**HOMEDICS®**

✓ Register your product today at  
[www.homedicsgroup.com/register](http://www.homedicsgroup.com/register)

# Eye Massager

**Instruction Manual**

EYE-100-EU

**2** year  
guarantee

READ ALL INSTRUCTIONS BEFORE USE.  
SAVE THESE INSTRUCTIONS FOR FUTURE REFERENCE.

## IMPORTANT SAFEGUARDS:

WHEN USING ELECTRICAL PRODUCTS, ESPECIALLY WHEN CHILDREN ARE PRESENT, BASIC SAFETY PRECAUTIONS SHOULD ALWAYS BE FOLLOWED, INCLUDING THE FOLLOWING:

- If you have any concerns regarding your health, consult a doctor before using this appliance.
- Only use this appliance for the intended use as described in this booklet.
- This appliance can be used by children aged from 14 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved. Children shall not play with the appliance. Cleaning and user maintenance shall not be made by children without supervision.
- This appliance should NEVER be used by any individual suffering from any physical ailment that would limit the user's capacity to operate the controls.
- Keep away from heated surfaces.
- Individuals with pacemakers should consult a doctor before using this appliance.
- DO NOT use on an infant, invalid or on a sleeping or unconscious person.
- DO NOT use on sensitive, very dry or damaged skin or directly on swollen or inflamed areas or skin eruptions. If you have circulatory or skin problems consult your doctor before using.

- Do not use on irritated, sunburned or chapped skin, or on any area with dermatological problems.
- If you experience discomfort when using this product, discontinue use. If a more extreme reaction occurs, consult your doctor.
- For external use only.
- This is a non-professional appliance, designed for personal use.
- Only use this appliance around the eyes as described in this booklet.


## BATTERY CAUTIONS

- Follow equipment instructions carefully and always use the recommended type of battery.
- For best results use AAA size LR6 Alkaline batteries.
- We do not recommend the use of rechargeable batteries.
- DO NOT put batteries in fire.
- DO NOT puncture, crush, dent or dismantle batteries.
- DO NOT attempt to revive primary batteries by heating, charging or other means.
- DO NOT allow children to fit batteries without adult supervision.
- DO NOT short circuit batteries by connecting the + and – terminals together.
- Keep batteries out of the reach of young children.
- Seek medical advice if you believe a battery has been swallowed.

## PRODUCT FEATURES:


### Fitting the Batteries

- This eye massager uses one AAA battery, preferably alkaline.
- Battery is supplied with this product.
- To fit the battery, remove the battery cover by sliding it away from the main unit.
- Insert the one AAA battery taking care to observe the correct polarity.
- Replace the battery cover by sliding it back onto the main unit.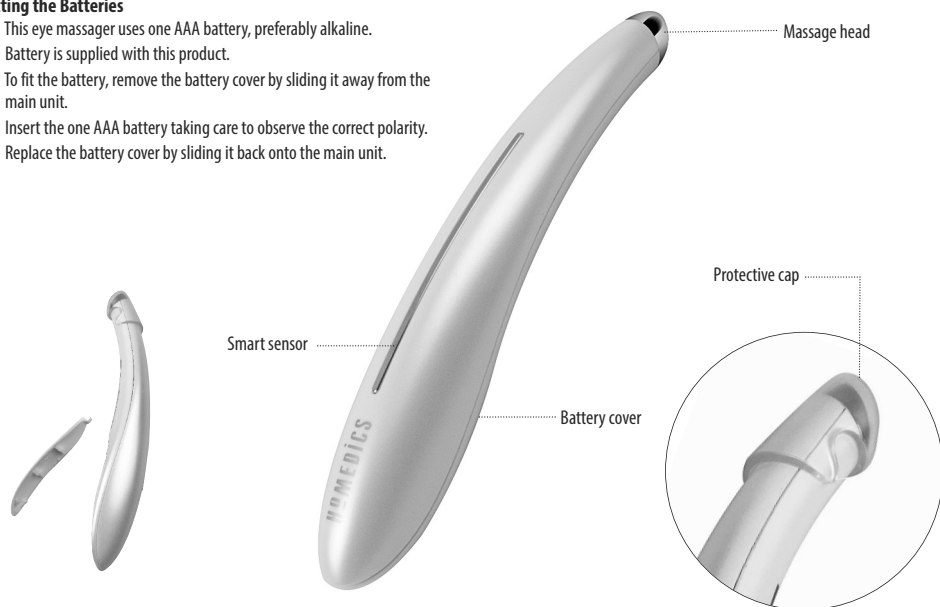

## USING YOUR EYE MASSAGER

1. Remove any eye make-up and apply a small amount of your favourite eye cream.
2. Remove the protective cap.
3. Hold the stem of the massager, making sure your skin is in contact with the smart sensor.
4. When the massage head is placed against your skin the unit will automatically turn on.  
**TIP:** If the massager doesn't turn on try applying a small amount of moisturiser to your hands.

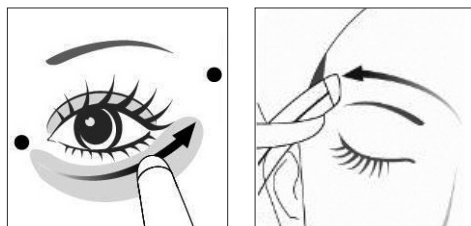5. Slide the massage head across your skin as shown, applying light pressure to give a gentle massage. When the correct level of pressure is applied the massager should smoothly glide across your skin, vibrating gently.
6. Clean the massage head and replace the protective cap after each use.

## MAINTENANCE:

Regular care and cleaning is essential to maintain your Eye Massager in peak working condition. Cleaning and user maintenance shall not be made by children without supervision. Regularly check the device for signs of damage.

### To Store

Store in a safe, dry place, out of children's reach.

### To Clean

Clean the unit after each use with a damp cloth and dry thoroughly before using. DO NOT use abrasive or chemical cleaners.

### Battery directive

 This symbol indicates that batteries must not be disposed of in the domestic waste as they contain substances which can be damaging to the environment and health. Please dispose of batteries in designated collection points.

### WEEE explanation

 This marking indicates that this product should not be disposed with other household wastes throughout the EU. To prevent possible harm to the environment or human health from uncontrolled waste disposal, recycle it responsibly to promote the sustainable reuse of material resources. To return your used device, please use the return and collection systems or contact the retailer where the product was purchased. They can take this product for environmental safe recycling.

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