HOMEDICS®

LEADERS IN BEAUTY & WELLBEING

BACK WAIST SUPPORT

Lumbar Support Belt

2 YEAR GUARANTEE

ER-BW100QVD

PRODUCT FEATURES:

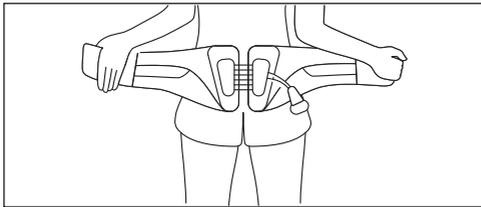
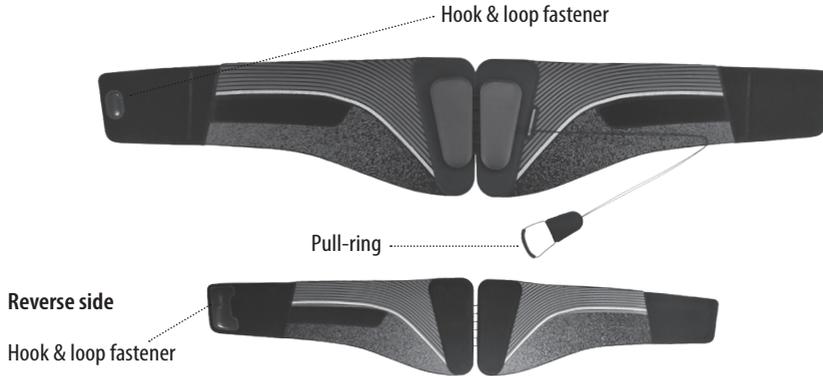
Fig. 1

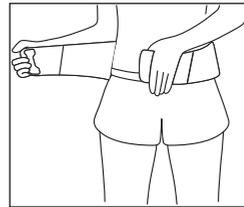
Fig. 2

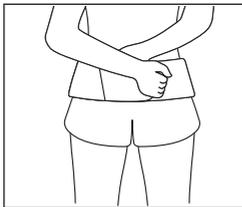
Fig. 3

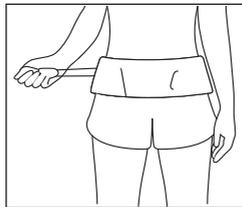
Fig. 4

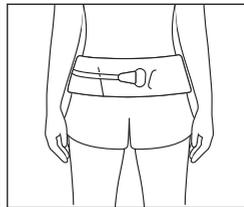
Fig. 5

INSTRUCTIONS FOR USE

For increased results and comfortable wear, please use the belt on top of your clothes.

1. Remove the pull-ring from the waist support and place the back waist support around your lower center waist. Make sure the pull-ring is on your right side (fig.1).
2. Wrap the back waist support around your waist, bringing the right side of the belt towards the left side (fig.2).
3. Pull tight and attach the hook & loop fastener on the right side of the belt to the left side (fig.3). If the waist support is too short to attach, add the adjusting belt provided.
4. Pull the pull-ring toward your body until you feel the belt start to restrain (fig.4).
5. Once comfortable, attach the pull-ring to the front of the belt using the hook & loop fastener (fig.5).
6. We recommend wearing the back waist support for a maximum of 2 hours at a time.

Maintenance

- Clean only with a soft, slightly damp sponge.
- Do not immerse in any liquid to clean.
- Never use abrasive cleaners, brushes, glass/furniture polish, paint thinner, etc to clean.

Storage

Keep the back waist support in a safe, dry, cool place. Avoid contact with sharp edges or pointed objects which might cut or puncture the fabric surface.

**READ ALL INSTRUCTIONS BEFORE USE.
SAVE THESE INSTRUCTIONS FOR FUTURE
REFERENCE.**

- This appliance can be used by children aged from 16 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved. Children shall not play with the appliance. Cleaning and user maintenance shall not be made by children without supervision.
- If you have any concerns regarding your health, consult a doctor before using this appliance.
- Use of this product should be pleasant and comfortable. Should pain or discomfort result, discontinue use and consult your GP.
- Pregnant women should consult a doctor before using this appliance. Not recommended for use by individuals with sensory deficiencies including diabetic neuropathy.
- DO NOT use on an infant, invalid or on a sleeping or unconscious person. DO NOT use on insensitive skin or on a person with poor blood circulation.
- Do not use on irritated, sunburned or chapped skin, or on any area with dermatological problems
- Do not use for longer than the recommended time.
- If you feel pain, swelling, or discomfort during use, discontinue use and seek medical attention immediately.
- Do not use on the neck or head

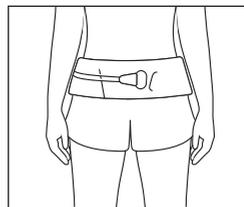
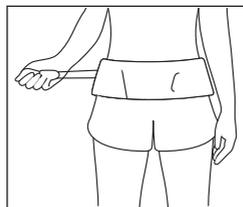
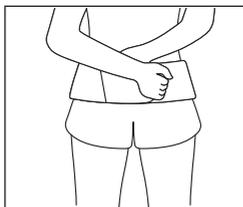
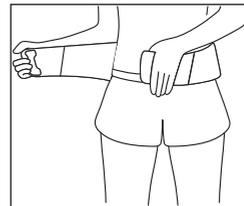
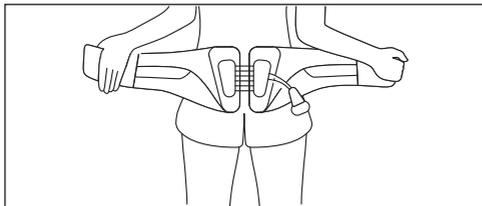
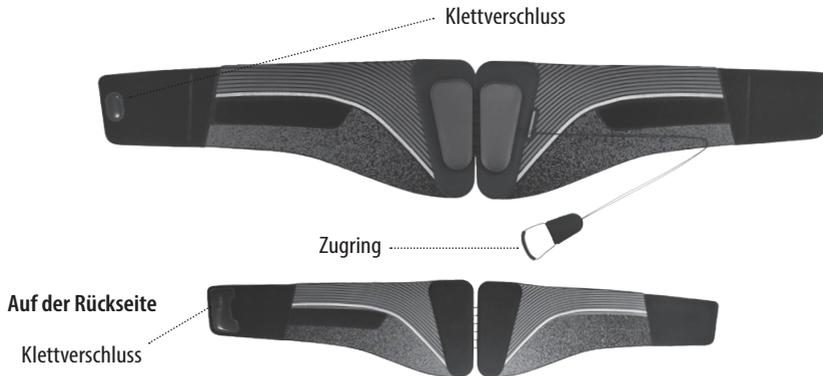
TWO YEAR GUARANTEE

FKA Brands Ltd guarantees this product from defect in material and workmanship for a period of 2 years from the date of purchase, except as noted below. This FKA Brands Ltd product guarantee does not cover damage caused by misuse or abuse; accident; the attachment of any unauthorised accessory; alteration to the product; or any other conditions whatsoever that are beyond the control of FKA Brands Ltd. This guarantee is effective only if the product is purchased and operated in the UK / EU. A product that requires modification or adaptation to enable it to operate in any country other than the country for which it was designed, manufactured, approved and / or authorised, or repair of products damaged by these modifications is not covered under this guarantee. FKA Brands Ltd shall not be responsible for any type of incidental, consequential or special damages.

To obtain guarantee service on your product, return the product post-paid to your local service centre along with your dated sales receipt (as proof of purchase). Upon receipt, FKA Brands Ltd will repair or replace, as appropriate, your product and return it to you, post-paid. Guarantee is solely through HoMedics Service Centre. Service of this product by anyone other than the HoMedics Service Centre voids the guarantee. This guarantee does not affect your statutory rights.

For your local HoMedics Service Centre, go to
www.homedics.co.uk/servicecentres

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Manufacturer & UK Importer: FKA Brands Ltd, Somerhill Business Park, Tonbridge, Kent TN11 0GP, UK
EU Importer: FKA Brands Ltd, 29 Earlsfort Terrace, Dublin 2, Ireland