GB

### WARNINGS:

- It is always important to follow these instructions and warnings on the use of the mask. If incorrectly used the mask cannot protect the wearer.
- A copy of the Declaration of Compliance can be found here: <https://www.homedics.co.uk/doc/>
- Always read the instructions before using the mask and always ensure the mask is in good condition and clean before use.
- Do not use the mask to enter or stay in a contaminated area under the following circumstances:
  - a. If the atmosphere contains less than 19.5% oxygen.
  - b. If the area is contaminated
  - c. For protection against gases or vapors.
  - d. If the contamination or concentrations of air are unknown or cause a danger to your life or health.
  - e. For sandblasting, paint-spray operations and asbestos treatment.
  - f. In an area with explosives

### N.B.:

- The mask marked "NR (not reusable)" is for single use only.
- Never substitute, modify, add, or omit parts of the mask
- The mask helps protect against certain contaminated particulates which are airborne but does not completely protect against the risk of contracting diseases or infections.
- If hair is trapped under the face seal, the mask will not be effective to protect against bacterial filtration
- Change the mask immediately if breathing becomes difficult or the mask becomes damaged
- Change the mask if it cannot be fitted correctly

### FITTING INSTRUCTION:

- Hold the mask upright in the correct position to cover your nose and mouth.
- Pull the head harness behind your ears and attach the head harness to the retaining clip. This improves comfort and prevents leakages.
- Ensure the nose clip is securely molded around the nose, resting the ends against the cheeks to obtain a good seal.
- To check if fitted correctly, cup both hands over the mask and exhale vigorously. If air leaks through around the nose; tighten the nose clip. If air leaks around the edge; reposition the head harness for better fit.
- Repeat the above adjustments until the mask is sealed properly.
- If a proper seal cannot be achieved, do not use the mask

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