**HOMEDICS®**

LEADERS IN BEAUTY & WELLBEING

# PHYSIO MASSAGE GUN

**DEEP TISSUE PERCUSSION MASSAGER**

**3 YEAR GUARANTEE**

PGM-200-EU

## PRODUCT FEATURES:

fig. 1

**Round massage head** suitable for massaging arms, waist, back, buttocks, thighs and other large muscle groups

**Conical massage head** suitable for targeting trigger points and small specific areas such as feet

**U-shaped massage head** suitable for trapezius muscles, areas around the achilles tendon muscle, ankle muscle and calf muscles

fig. 2



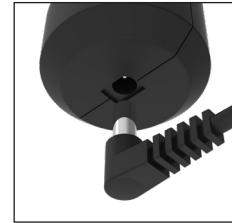
fig. 3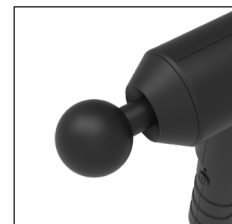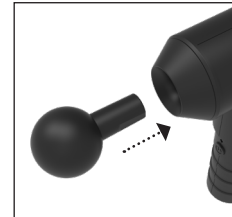

## INSTRUCTIONS FOR USE:

### Charging your device

1. To charge the product, plug the charging cable into a USB socket and connect the cable to the charging socket on the bottom of the handle (fig.2).
2. Once the charging cable is connected, the charging indicator LED will illuminate and blink red. As soon as the device is fully charged the LED indicator will change to blue.
3. Once the product is fully charged, disconnect it from the power supply.
4. The product requires 5-6 hours of charging for a full charge and will last for approximately 1 hour of use.

### Using your device

1. Select the desired massage head and fasten it into the socket on the front of the massager (fig.3). It should be finger tight only as overtightening may cause damage to the product.
2. Press and hold the power on/off/power level button for 3 seconds to turn the device on.
3. Pressing the on/off/power button again will cycle through the power levels:
  - Press x 1: **LO**
  - Press x 2: **MID**
  - Press x 3: **HI**
  - Pressing the button again will repeat this sequence.

4. Gently move the massage head over the part of the body you wish to massage at first, then gently apply more pressure as desired. If you are a first time user it is recommended that you start on power level 1 and press gently.
5. A long press on the on/off/power button for 1 second will turn the massager off.
6. The product cannot be used while charging

**DO NOT USE THE PRODUCT FOR OVER 30 MINUTES CONTINUOUSLY. IF THE PRODUCT IS USED FOR OVER 30 MINUTES IT MAY TRIGGER THE HEAT PROTECTION FEATURE AND STOP AUTOMATICALLY. PLEASE WAIT FOR AT LEAST 30 MINUTES FOR THE PRODUCT TO COOL OFF BEFORE RE-USING.**

### Cleaning your device.

- Make sure the device is unplugged from any power supply and allow to cool before cleaning. Clean only with a soft, slightly damp sponge.
- Never allow water or any other liquid to come into contact with the appliance.
- Do not immerse in any liquid to clean.
- Never use abrasive cleaners, brushes, glass/furniture polish, paint thinner, etc to clean.

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fig. 1

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**FKA Brands Ltd**