# HOMEDICS®

LEADERS IN BEAUTY & WELLBEING

# CORDLESS DOUBLE-BARREL

RECHARGEABLE BODY MASSAGER

**3 YEAR GUARANTEE**

SP-180J-EU2

Register your product today at  
[www.homedics.co.uk/product-registration](http://www.homedics.co.uk/product-registration)

## PRODUCT FEATURES

Fig. 1

Fig. 2

Fig. 3

Fig. 4

Fig. 5

## INSTRUCTIONS FOR USE:

1. Your unit should arrive with a full charge. When you need to charge the unit, plug the adapter into the jack on the unit, and plug the other end into a 100-240V mains outlet. The power button will illuminate red when charging and will change to green once fully charged. The unit should be charged after 5 hours of charge time. A full charge will last up to 2 hours.

**Note:** Product should only be charged using the adaptor supplied (SAW06C-050-1000GB). Adaptor must be removed from the socket when not in use. Adaptor output charging voltage of 5Vdc and 1A must not be exceeded.

2. This massager is versatile and can be used for neck, shoulder, back, legs, arms and feet (Fig. 1-3). For use on neck, shoulders, or back, attach the adjustable straps to the ends of the unit (Fig. 4), use the straps to hold the massager in the desired area. The double-barrel design allows the massager to roll up and down your body relaxing muscles and alleviating pain as it rolls.
3. To activate the massage action, short press the power button (Fig. 5) and the vibration waves will start on the lowest setting. Press 2 seconds to medium intensity and press 2 seconds again to experience the highest intensity. To turn the unit off, short press and turn off.
4. When not in use, store your Cordless Double-Barrel Massager in its convenient drawstring storage bag.

### Cleaning

Unplug the appliance and allow to cool before cleaning. Clean only with a soft, slightly damp sponge.

Never allow water or any other liquids to come into contact with the appliance. Do not immerse in any liquid to clean.

Never use abrasive cleaners, brushes, glass / furniture polish, paint thinner etc to clean.

### Storage

Place the appliance in its bag or in a safe, dry, cool place. Avoid contact with sharp edges or pointed objects which might cut or puncture the fabric surface. To avoid breakage, DO NOT wrap the power cord around the appliance. DO NOT hang the unit by the cord.

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**FKA Brands Ltd**
Manufacturer & UK Importer: FKA Brands Ltd, Somerhill Business Park, Tonbridge, Kent TN11 0GP, UK
EU Importer: FKA Brands Ltd, 29 Earlsfort Terrace, Dublin 2, Ireland