# HOMEDICS®

LEADERS IN HEALTH & WELLBEING

✓ Register your product today at  
[www.homedics.co.uk/product-registration](http://www.homedics.co.uk/product-registration)

## SHIATSU MASSAGE PILLOW WITH SOOTHING HEAT

instruction manual

SP-7H-EU

3 YEAR guarantee

## PRODUCT FEATURES

### Massage Button

Massage ON/OFF

### Heat Button

To turn the soothing heat function on, first turn on massage. Then press the soothing heat button once to activate soothing heat. Press the soothing heat button again to stop soothing heat function.

### Shiatsu Massage

Shiatsu is a deep kneading circular massage. This unit features a spot Shiatsu massage mechanism that works on various areas of your body

**NOTE:** There is a 15-minute auto shut off on the unit for your safety.

## INSTRUCTIONS FOR USE:

Always ensure the appliance is set to the 'OFF' before inserting the plug into a mains outlet.

1. Plug the appliance into a 220-240V mains outlet and switch on.
2. To activate the Shiatsu massage action, press power button  once (Fig. 1). To activate soothing heat, press heat button  below the power button (Fig. 1).
3. When finished press the power button  again to turn off the massage function and unplug the pillow.

Fig 1

Fig 2

For the neck and shoulders it may work better to use the pillow upside down.

The Shiatsu Pillow is very versatile and designed for use on the neck, back and shoulders (Fig. 2). However, you may find it works well for other body areas as well.

### NOTE:

- DO NOT attempt to stand on this Foot Massager. Use only while seated.
- The Heat function will only work when the massage is in use.
- DO NOT use the massage or heat functions continuously for more than 15 minutes.

## MAINTENANCE:

### Cleaning

Unplug the appliance and allow to cool before cleaning. Clean only with a soft, slightly damp sponge.

- Never allow water or any other liquids to come into contact with the appliance.
- Do not immerse in any liquid to clean.
- Never use abrasive cleaners, brushes, glass / furniture polish, paint thinner etc to clean.

### Storage

Place the appliance in its box or in a safe, dry, cool place.

To avoid breakage, DO NOT wrap the power cord around the appliance. DO NOT hang the unit by the cord.

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**FKA Brands Ltd**