HOMEDICS®

LEADERS IN BEAUTY & WELLBEING

THERMAL PRO

DEEP TISSUE PERCUSSION MASSAGER

PGM-300BKQVD-EU
PGM-300RGQVD-EU
PGM-300WTQVD-EU

PRODUCT FEATURES:

fig. 1

Heat

Cool

fig. 2

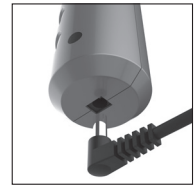
fig. 3

fig. 4

USB charging cable

Adaptor

Storage bag

MASSAGE HEADS

HEAT Active thermal head relaxes tired aching muscles

COOL Cooling calms inflamed stressed muscles

INSTRUCTIONS FOR USE

Charging your device

1. To charge the product, plug the charging cable into a USB socket and connect the cable to the charging socket on the bottom of the handle (fig.2).
2. Once the charging cable is connected, the charging indicator LED will illuminate and blink red. As soon as the device is fully charged the LED indicator will change to blue.
3. Once the product is fully charged, disconnect it from the power supply.
4. The product requires 5-6 hours of charging for a full charge and will last for approximately 1 hour of use.

WARNING If the voltage input exceeds 5V, the massager will terminate charging and the indicating light will blink in blue in order to remind the user to use the correct voltage adaptor. Product should only be charged using the adaptor supplied (DJ0500100-511EU). Adaptor must be removed from the socket when not in use. Adaptor output charging voltage of 5Vdc and 1A must not be exceeded.

Using your device

NOTE Remember to switch off the heat when you have finished using by pressing the heat button (fig.4).

1. Select the desired massage head and fasten it into the socket on the front of the massager (fig.3). It should be finger tight only as overtightening may cause damage to the product.
2. Press and hold the power on/off/power level button for 3 seconds to turn the device on.
3. Pressing the on/off/power button again will cycle through the power levels:
 - Press x 1: **LO**
 - Press x 2: **MID**
 - Press x 3: **HI**
- Pressing the button again will repeat this sequence.

4. To use heat treatment with Heat massage head, press the heat button on the back of the product once (fig.4). The red LED light ring on the Heat head will illuminate. Press the power button to start the massage and adjust intensity as above. **NOTE** Heat function only works with Heat massage head.
5. **IMPORTANT** Before using the Cool massage head, place in a freezer for several hours.
6. Gently move the massage head over the part of the body you wish to massage at first, then gently apply more pressure as desired. If you are a first time user it is recommended that you start on power level 1 and press gently.
7. A long press on the on/off/power button for 1 second will turn the massager off.
8. The product cannot be used while charging

DO NOT USE THE PRODUCT FOR OVER 30 MINUTES CONTINUOUSLY. IF THE PRODUCT IS USED FOR OVER 30 MINUTES IT MAY TRIGGER THE HEAT PROTECTION FEATURE AND STOP AUTOMATICALLY. PLEASE WAIT FOR AT LEAST 30 MINUTES FOR THE PRODUCT TO COOL OFF BEFORE RE-USING.

Cleaning your device

- Make sure the device is unplugged from any power supply and allow to cool before cleaning. Clean only with a soft, slightly damp sponge.
- Never allow water or any other liquid to come into contact with the appliance.
- Do not immerse in any liquid to clean.
- Never use abrasive cleaners, brushes, glass/furniture polish, paint thinner, etc to clean.

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TECHNICAL SPECIFICATIONS

Battery Capacity	3.7V/4000MAH(14.8W)
Charging voltage	5V USB 1A
1st mode Speed	2800 Rpm 46 Hz
2nd mode Speed	3400 Rpm 56 Hz
3rd mode Speed	3900 Rpm 65 Hz
Charging Time	5-6 hours
Work for when full charge	1 hours
Auto Timer	10 minutes
Weight only the massager	682 gms / 1.50 lbs
Overall size of massager	23.5 x 6 x 19 cm

Manufacturer & UK Importer: FKA Brands Ltd, Somerhill Business Park, Tonbridge, Kent TN11 0GP, UK
EU Importer: FKA Brands Ltd, 29 Earlsfort Terrace, Dublin 2, Ireland
Customer Support: +44(0) 1732 378557 | support@homedics.co.uk

IB-PGM300BKQVDEU-0920-01
IB-PGM300RGQVDEU-0920-01
IB-PGM300WTQVDEU-0920-01